Ultimate Comfort

Modelo: Smart

C20-0094
1801ES-ES-EN

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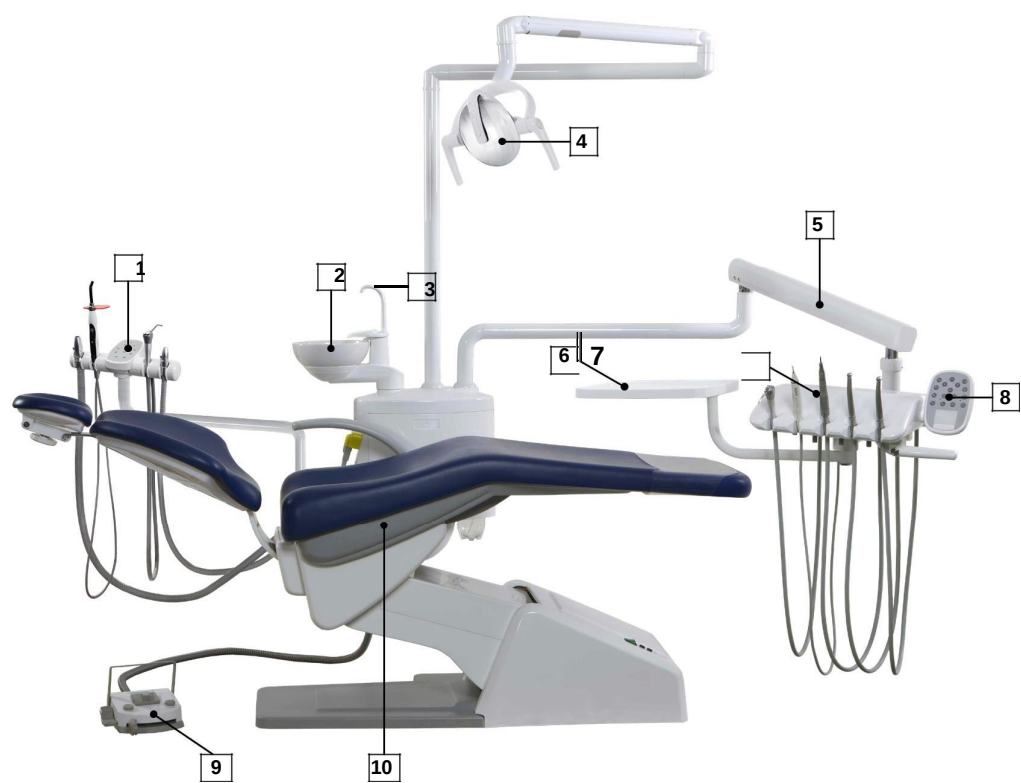
Dear user!

Our congratulations on the choice of your new Ritter product and many thanks for your confidence!
This modern device was produced by our development- and production engineers with great carefulness.

You and your team will gain absolute joy and benefit of your new Ritter device, if you and your employees, after proper mounting on site, follow the technical and application-oriented advices from the handing over demonstration.

Despite of a careful introduction into the operation, smaller or greater questions will arise in the course of time. This operating manual shall give you answers to these questions. To make day-to-day work easy for you and to be sure, that all technical or legal regulations and the maintenance notes receive their attention, we recommend a careful examination of this manual for you and your responsible employees too.

If you face technical problems, our service hotline
+49 (0) 37754/13-290 is available for you.
And now a lot of joy with your Ritter equipment.

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No.	Description
1	Assistant element
2	Spittoon bowl
3	Cup filler
4	Operating lamp
5	Table arm
6	Tray
7	Dentist element
8	Dentist keyboard
9	Multifunctional foot controller
10	Patient chair

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General

Foreword	3
Overview	4
Table of content	5
Technical data	6
Operating/transport and storage conditions	7
Signs and symbols	8
Intended use/General information	10
Safety information	12
Working area.....	13
Notes	14

Dentist element

Device description	16
Features	17
Bedienelemente.....	18
Control elements	18
Foot controller	19
Arm brake.....	20
Regulation of spray-water	22
Air pressure regulation, Prompt tone	23
Ultrasonic scaler	24
Syringe	25
Operating lamp „Standard Light LED“	26
Operating lamp „SunLite“ (optional)	28

Assistant element

Operation assistant element	32
Spittoon bowl rinsing/Cup filler/Heating.....	34
Suction/Aqua bottle system	35
Aqua bottle system	36

Patient chair

Working area.....	38
Manual control	39
Program control	40
Headrest	42

Maintenance

Daily	44
Troubleshooting	51
Self test, Factory settings.....	53

Appendix

Setting up drawing.....	56
Air-water-schematic	57
Wiring diagram	58
Safety inspection.....	60
EMC declaration	61
Terms of Guarantee.....	63

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Dental work place

Type:	Ultimate Comfort
Model:	Smart
Rated voltage:	220 V
Frequency:	50/60 Hz ± 1 Hz
Rated input:	900 VA
Fuse:	10 A time lag
Protection class:	I
Type of applied part:	B
Heater:	24 V AC, 120 VA
Degree of water proof:	Non-water-tight
Mode of operation:	DAB*
Weight:	255 kg
Dimensions:	190 x 85 x 180 cm (l x w x h)
Rotation range of the spittoon:	≥180°

Air source:	
Air pressure:	≥5.5 bar (≥550 kPa)
Air flow:	≥50 l/min

Water supply:	
Water pressure (input):	2.0...4.0 bar**
Water flow:	≥10 l/min
Requirements for water quality:	
Optimum of water hardness:	1.42 ... 1.78 mmol/l (8 ... 10 °dH)

Water unit

Rotational angle of spittoon bowl:	180°
------------------------------------	------

Patient chair

Mode of operation:	DAB* (2 min/18 min)
(Intermittent Duty Time Rating:	On: 2 min Off: 18 min)

Maximum height (seat cushion floor):	800 mm
Minimum height (seat cushion floor):	380 mm

Maximum stroke:	400 mm
Backward angle of backrest:	102°-192°
Up and down sliding stroke of head rest:	200 mm
Maximum loads:	200 kg

Dentist element:

vertical motion range	
of the instrument table:	480 mm

Dental operating light „Standard Light LED“

Type:	LED
Rated voltage:	12- 17 V ~ 50/60 Hz
Rated input:	6 VA
Mode of operation:	DAB***
Protection class:	II
Illuminance	8000-30000 lx (in distance of 700 mm)
Colour temperature:	5000 K
Light spot:	8 cm x 15 cm
Lifetime cycle:	5 years

Dental operating light „SunLite“ (optional)

Type:	LED
Rated voltage:	17- 24 V ~ 50/60 Hz
Rated input:	20 VA
Mode of operation:	DAB***
Protection class:	II
Illuminance	8000-35000 lx (in distance of 700 mm)
Colour temperature:	4200-6000 K
Colour rendering index (CRI):	>85
Lifetime cycle:	50000 h

Technical modifications reserved!

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DAB*=continuous operation with intermittent load. The permissible load times are adapted to the working mode of the dentist (please refer also to the operating manual for the individual units and/or instruments).

**=At site there must be a back-flush protective filter (mesh size 80-120 µm) for the water inlet pipe to the supply unit. The complete installation must be according to the official standard EN 1717 conducted.

DAB***= continuous operation

Operating conditions

Temperature: +10°C to +40°C
 rel. humidity: 30% to 75%
 Barometric pressure: 700 hPa to 1060 hPa

Transport and storage conditions

(original packing)



Temperature: -20°C to +70°C
 rel. humidity: 10% to 90%
 Barometric pressure: 500 hPa to 1060 hPa

Explanations to name plates

Icon	Explanations to type plates
	Serial number
	<u>CE-Approval</u> according to Directive 93/42/EEC for medical products
	The symbol of the crossed waste container on wheels indicates, that Ritter products are electrical equipment according to Directive 2002/96/EC. Within the EU-Countries these products have to be separately disposed. Ritter wants to encourage you, to minimize possible effects of waste to environment and health outside of the EU, too. Please follow the local directives for disposal of waste and make use of the separate collection of electrical equipment.
	There are specific warnings or precautions related to this product.
	Type B applied part
	Following the instruction for use
	Manufacturer/Date of manufacturing (according EN 980 remark 2)

The name plate is posted at the bottom of the patient chair.

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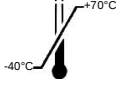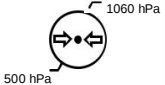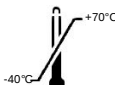
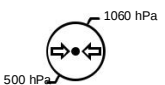
Icons on packaging

Icon	Explanations to type plates
	Fragile, Handle with care
	This way up
	Do not stack
	Keep dry
	Permissible temperature range
	Permissible range of humidity
	Permissible range of barometric pressure
	<u>CE-Approval</u> according to standard 93/42/EEC for medical products
	Serial number

Icons in this Instruction for use

Icon	Legend (used icons in this manual)
	Note
	Attention Observe special precautions or warnings for this product.

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Our equipment complies with the requirements of standards for medical products (MPG) and Directive 93/42/EEC for Medical Devices, revised by EU-Directive 2007/47/EG from 2010-03-21.

It complies with the VDE-requirements for electromedical devices DIN EN 60601, part 1 / DIN VDE 0750, part 1, too. It is suited for connection to supply lines installed according to the DIN VDE 0100-710 regulations for medically used rooms.

Intended use

This dental treatment unit is used for dental treatment in humans by qualified personnel.

Warnings

	This unit is not explosion-proof.
	This dental treatment unit must not be modified without permission of the manufacturer.
	We can only accept responsibility for the safety properties of our equipment, if installation, maintenance, repair and modifications are carried out by our workshop or by a person explicitly authorized by us. Further, the equipment has to be correctly handled according to our operating manual.
	In case of failure, structural parts, that can affect the security of the device, are to be replaced by original parts only.
	In case of repair, the technician must issue a certificate as to the kind and extent of work performed and, if applicable, indicate any changes of nominal data and operating range. The certificate must contain the date of repair, the name of the executing company and a signature (according to MPG, DIN VDE 0750 / DIN VDE 0751).
	Close to the equipment do not operate any devices producing electro-magnetic emissions (such as wire-less phones, micro-wave therapeutics devices,...)

Safety-relevant inspections

The safety standards for medical products and all legal obligations are to be fully observed by the user.

Scope and schedules of safety-relevant inspections are as follows:

Every two years the following safety-relevant inspections according to DIN EN ISO 62353 have to be performed:

see Form „Safety inspections – Dental treatment unit“.

We recommend to keep an equipment control log for documenting the results of the safety-relevant inspections.

The unit lifetime cycle is: 10 years.

	Operator (intended user) The user must have a completed medical training (study), specializing in dental, or have comparable field of study.
	Multimedia For components like Intraoral camera, monitor, note the separate user manuals for these devices!

Disposal

According to law, electric equipment is to be disposed by authorized disposal firms only.

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Please always follow the following safety precautions to avoid injuries to persons or damages to the unit:

	WARNING!
	Please always shut off the water supply, air supply and the main power before performing any cleaning or maintenance.
	Please do not place the unit on shaky floor, cart and shelf or it may tumble over and badly damaged.
	Please do not install the unit near radiators or heaters.
	Please do not place or hang any heavy things on the unit.
	Please do not connect unit with incompatible wires which may affect its performance, or even cause fire or electric shock.
	Please do not place anything in the unit so as to avoid electric shock or fire. Please do not spill any liquid into the inner part of the unit.
	Please do not disassemble the unit so as to avoid electric shock. Contact technical service personnel whenever the unit needs repairing.
	Please make sure to use earthed plug. Bad earthing may cause electric shock to the user or damage to the unit.
	Please do not open or remove the cover plate so as to avoid injuries caused by dangerous voltage or other dangers. An improper assembly may cause electric shock during the subsequent operation.
	Please switch off the power and seek help from technical service personnel if: (1) any break-down or wear-down of the plug or cable occurs. (2) unknown liquid spills into the inner unit or water flows into the inner circuits. (3) the unit malfunctions, even if it is operated according to the instructions.
	If possible, please turn off the power supply during thunderstorm.
	Please never exceed this limit (maximum load of the dental seat: 135 kg).
	The maximum load of the table tray attached to the dentist's instrument board is 3.0 kg, which should never be exceeded.
	The manufacturer shall not be responsible for any damage or injury caused by mishandling, neglect or improper use.
	Before leaving the surgery, shut off the water supply, air supply and the main power.

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Translation of user manual

When imported into countries whose native language is not English or German, the operating manual are translated into local languages. In this translation, replace the words „original instruction for use“ by the following notice: „translation of the original instruction for use.“

The Ritter® Concept GmbH is not directly or indirectly liable for mistakes or failures in this documentation as well for the given benefits of the use with this operating instruction!

The illustrations in this manual may be able to differ from the actual unit in your system.

The duplicating of this documentation without written permission of Ritter® Concept GmbH is prohibited!

Recycling

All packing material does not harm the environment and can be completely recycled.

- wooden base plate
- carton box
- polyethylene bag

Features and options – Stools

- Mobilo Flex 50
- Mobilo Flex 50 with footring (optional)
- Backrest with fixed position or dynamic movement
- Ergonomic and comfortable seating
- Symmetrical backrest
- Gas lift cylinder
- Chrome legs
- Five dual castors for stability
- Colour-matching upholstery available in multiple colours

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We recommend you to read this operating manual very carefully in order to have full knowledge of all the advantages this dental workplace is offering you.

The essential attributes of all models are:

- 5 outlets for turbine or air motor
- cut off air/water switch for safety during changing drills

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Model "H" (hanging hoses)

- instrument carrier with hanging hoses
- smooth running swivel arm
- pneumatic limited
- integrated tray
- photoelectric sensor in Instrument holder

Model "S" (swinging hoses)

- Instrument carrier with swinging lever
- smooth running swivelarm
- pneumatic limited
- photoelectric sensor in Instrument holder
- maximum rotation angle of instrument tray: 180°

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Startup

The unit is activated via the power switch (1) at the patient chair pedestal. After switching-on the control indicator of the power switch lights.

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For start up the unit switch on the main air break switch (2) that is located at the bottom of the dentist element. (main air break switch in position „Open“) It is recommendable to actuate the hand pieces, the cup filler and the spittoon bowl rinsing before the first daily treatment is done in order to rinse out stale water.

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For safety reasons, switch off the main switch when leaving the surgery! Therefore press main switch (1). After switching-off, the control indicator of the power switch doesn't light. Using stop cocks to shut off air and water inlet.

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If suddenly power outage switch off the main switch (1) and take all handpieces in their instrument holder. When electrical power comes back activate the unit via the power switch (1).

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Features and options – foot controllerfor chair movement and instruments:

- large foot bow for easy repositioning
- manual control of chair movements

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Program control of chair movements:

- Spitting position/last treatment position
- three free programmable chair positions
- Exit position

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Control of instruments:

- Speed/power control for the active instrument

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Buttons:

- Spray water at instrument
- Chip blower function

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Control elements patient chair	
A	Foot lever
If all instruments (except the syringe) are in their holders.	
2	Slide tip switch to the <u>left</u> Backrest DOWN Slide tip switch to the <u>right</u> Backrest UP Slide tip switch to the <u>backwards</u> Patient chair UP Slide tip switch to the <u>forward</u> Patient chair DOWN
3	press key <u>shortly</u> : Automatic movement to „Spitting position“, actuating again moves chair into last treatment position press key longer than <u>2 seconds</u> an acoustic signal sounds, Automatic movement to programmed „Position A“
4	press key <u>shortly</u> : Automatic movement to „Exit position“ press key longer than <u>2 seconds</u> an acoustic signal sounds, automatic movement to programmed „Position B“

Control elements for instruments	
3	press key <u>shortly</u> : activates the spray water at instruments
4	with pressed key activates the chip blower-function
5	push lever to the right: starting the activated instrument and increase the speed/power
6	maximum speed/power of the instrument
7	Press the pedal. Start the active instrument and increase/decrease the speed/power

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Arm brake

Moving the dentist element:

- touch the dentist element on the handle (2)
- press with thumb the button (1) and move dentist element in the desired position
- release button (1)

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X-ray viewer

- Press the key „X-ray viewer“ to activate the X-ray viewer.
- It stops automatically after 3 minutes.
- Premature power off the X-ray viewer can be released by pressing key "X-ray viewer" once more.

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Whip arm (dentist element S version)

- take up an instrument and pull it forwards till a click sounds
→ then the instrument is ready for working
- after using, swing back the instrument on the instrument holder
(after a click sounds the instrument is deactivated)

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Startup:

Take the hand piece from its holder.

The hand piece is activated via the lever (1) or pedal (2) of the footcontroller.

Depending on position of the lever (A) or the pressure of the pedal (B) – the power of the taken hand piece changes from a minimum to the maximum power.

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Spray water

To switch the spray water at instrument ON/OFF: press left pushbutton.

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Chip blower:

- switches an constant air flow at the activated instrument ON or OFF
- As long as pressed the right pushbutton - the chip blower function is activated.

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Instrument light:

- switches the light at the instrument ON or OFF (key at dentist keyboard)

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
Setting the maximum rotation speed of micromotor

The regulator (1) is at the backside of the dentist element.

Reversal of rotation direction:

- switches micromotor into anti-clockwise rotation
- The regulator (2) is at the backside of the dentist element.

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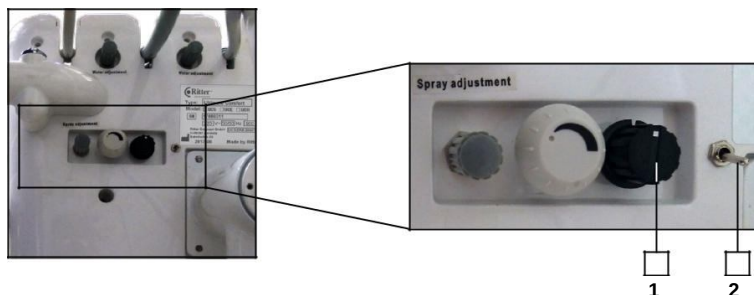
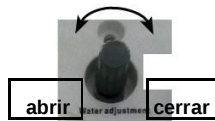
For each instrument (except the syringe) there is the possibility to regulate the spray water quantity by hand.

The regulators can be found at the backside of the dentist element (1).

The order of regulators depends on the order of the instruments on the tableau.

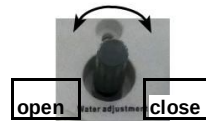In order to regulate the water, take the according instrument from its holder and actuate it with spray water function. Now you can adjust the water quantity by turning the according regulator as illustrated in the picture below.

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Spray regulator for all instruments

The regulator (2) is at the backside of the dentist element.

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Air and Water pressure adjustment of handpiece:

There are air and water regulating valves used for adjusting the water vapor.


1	Interruptor de aire (para toda la unidad)	air break switch (for whole unit)
2	Regulador de aire Para 3 instrumentos rotatorios	air pressure regulator for 3. th burr instrument
3	Regulador de aire para 2 instrumentos rotatorios	air pressure regulator for 2. nd burr instrument
4	Regulador de aire Para 1 instrumento rotatorio	air pressure regulator for 1. st burr instrument

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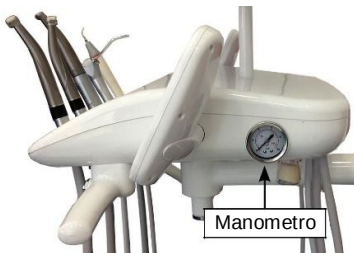
Adjustment drive air for handpiece:

Low speed pneumatic handpiece

No-load speed

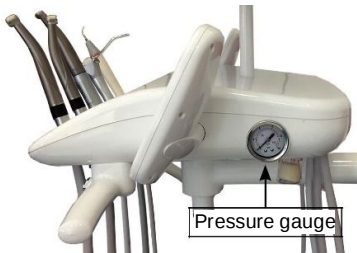- shall be no lower than 14000 min⁻¹ if the air pressure is 2.5 bar (250 kPa, impelling air)

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High speed turbine

No-load speed

- shall be no lower than 30000 min⁻¹ if the air pressure is 250 kPa (impelling air)
- a air pressure regulator is at the bottom of the dentist element
→ reduce the air pressure by rotating clockwise the regulator (2), (3) or (4)

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Enable/disable the prompt tone function

- press key  longer than 5 seconds to enable/disable the prompt tone function
→ a buzzer beep sounds
→ the prompt tone function of the key is enabled/ disabled

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Scaler

Startup:

Take the hand-piece out of the holder.

The device is activated with the lever (A) or pedal (B) of the foot controller.

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Intensity

The intensity of the scaler can be adjusted in by the regulator (3) at the backside of the dentist element.

Increase the power:

→ rotate the regulator (2) anti-clockwise

Reduce the power

→ rotate the regulator (2) clockwise

•

The scaler can not be regulated with the foot controller.

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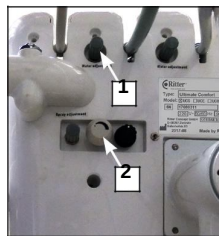
→ Vea página 22

Adjust water volume of scaler

• by using the corresponding water regulator (1) at the backside of the dentist element

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Note maximum power settings for using scaler tips!

→ see notes in instruction for use of scaler.

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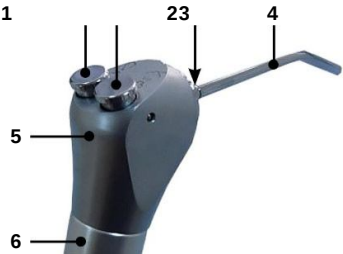
Functions:

Water, air or spray are switched by hand. Supply of the media lasts as long as the corresponding key of handle is pressed. Either air or water can be obtained by actuating the corresponding key. Actuating both keys at the same time will supply spray.

Operation of 3F syringe:

- 1 Key for water delivery
- 2 Key for air delivery
- 1+2 Delivery of spray

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1	Key for water delivery
2	Key for air delivery
1+2	Delivery of spray
3	Spigot ring
4	Cannula
5	Three-way Syringe
6	Handle

 Removable and autoclavable cannula (4).
Sterilization of cannula:
→see page 48 (chapter Maintenance)

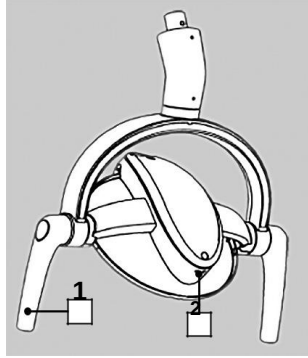
Fig.: Luz operativa „Standard Light LED“
Fig.: Operating lamp „Standard Light LED“

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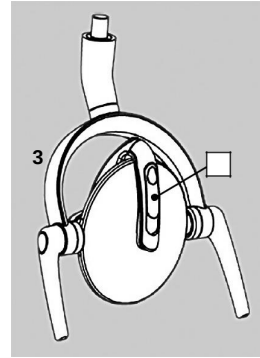
Operating light is fixed on flexarm mounted to the unit.

Essential features:

- comfortable, shadow less light
- easy flexible, keeps the adjusted position safely
- removable handle, sterilizable
- second light modus with yellow light (working with photosensitive resin)
- color temperature: 5000 K

Control elements:

1	Handle
2	Sensor

Startup:

The operating light can be turned on and off with the key „Operating lamp“ at dentist or assistant keyboard. Short press the button .

To switch on and off, move the hand near to the sensor (2) to a distance about 5 to 12 cm.

Light intensity

Increase the light intensity

- Place the hand near the sensor (2) until desired intensity is reached.

(Luminance increase from minimum to maximum occurs in 4 steps. After reaching the maximum luminous intensity, the lamp switches back to minimum light intensity.)

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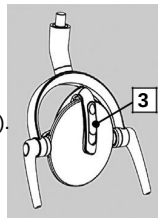
Blue cut-off function:

The dental operating light has blue cut-off function.

While using photosensitive resin to fill the teeth, adjust the color temperature of operating light into a yellow light (without blue light emission). So it is possible to prevent fast curing of photosensitive resin.

**Standard light (at a regular oral treatment):**

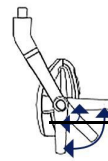
- move sliding switch (3) **up** and lock it properly

Yellow light (treatment with photosensitive resin):

- move sliding switch (3) **down** and lock it properly

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**Adjust operating lamp**

Three active joints are available on the dental operating light which can adjust the desired angle.

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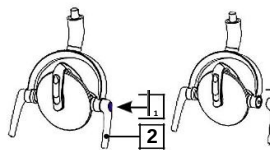
Change operating lamp handle**Remove handle**

- hold the operating light with the left hand
→ hold handle (2) with the right hand.
- press the handle button (1) with the thumb and pull out the handle

Install the handle

- hold the operating light with the left hand
hold handle (2) with the right hand
- press the handle button (1) with the thumb and insert the handle (2) to the spindle slot and loosen the handle

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The clearly geometry of reflectors, in combination with each LED source is, allows to realize a luminous flux which is homogeneous, clean and without shadows.

Essential features:

- comfortable, shadow less light
- easy flexible, keeps the adjusted position safely
- removable handle, sterilizable
- dimmer
- noiseless fan
- adjustable color temperature

Control elements:

1	Handle
2	Regulator for brightness
3	Regulator for Colour temperature

Startup:

The operating light can be turned on and off with the key „Operating lamp“ at dentisr or assistant keyboard. Short press the button .

Light intensity

The required intensity can be adjusted individually by turning the dimmer (2).

Reduce light intensity

- Rotate the brightness regulator (2) anti-clockwise (rear view of the operating lamp) until it reaches the desired light intensity.

Increase the light intensity

- Rotate the brightness regulator (2) clockwise (rear view of the operating lamp) until it reaches the desired light intensity.

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Adjustable range: 4.200 K - 6.000 K

Increase the colour temperature:

→ rotate the regulator (3) anti-clockwise

Reduce the colour temperature:

→ rotate the regulator (3) clockwiseLight spot

Size: 7 cm x 14 cm at a distance of 70 cm far from the source and with well defined outlines, in order to avoid risk of patient dazzling.

Each time the lamp is turned on, the light intensity will be at the level memorised when it was turned off the time before.

Patients at risk (i.e. with eye conditions) are not allowed to look into the light beam.

WARNING - reversible dazzling with wrong use of LED operating lamp -

A wrong use of the LED operating lamp can lead to temporary visible disturbance, therefore, when turning on or moving of lamp, never direct light field to eyes of patient, yourself and others. Always ensure a distance of approx. 70 cm between lamp and head of patients.

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Features and options – Assistant element

- moveable assistant element with assistand keyboard
- 3 Functional syringe
- big and small suction hose
- Polylight LED (optional)

Features and options - water unit

- swivel ceramic cuspidor
- ABS Aqua Bottle System
- Heating for cup filler
- Connection for external suction (optional)
- Place selection valve (optional)
- Venturi suction system

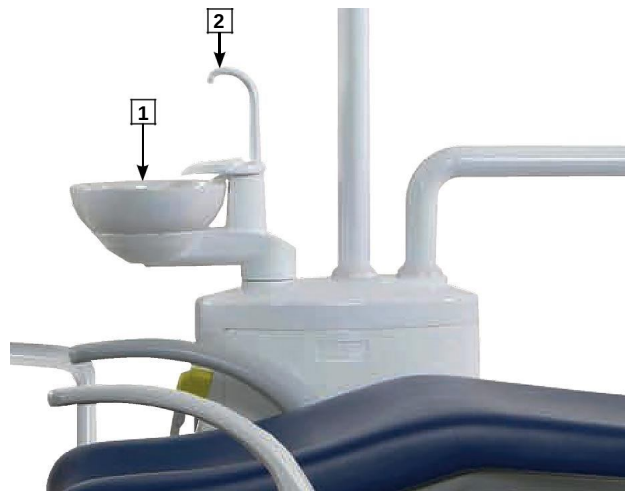
Fig.: escupidera (1), llenado de agua (2)
Fig.: Spittoon bowl (1), Cup filler (2)

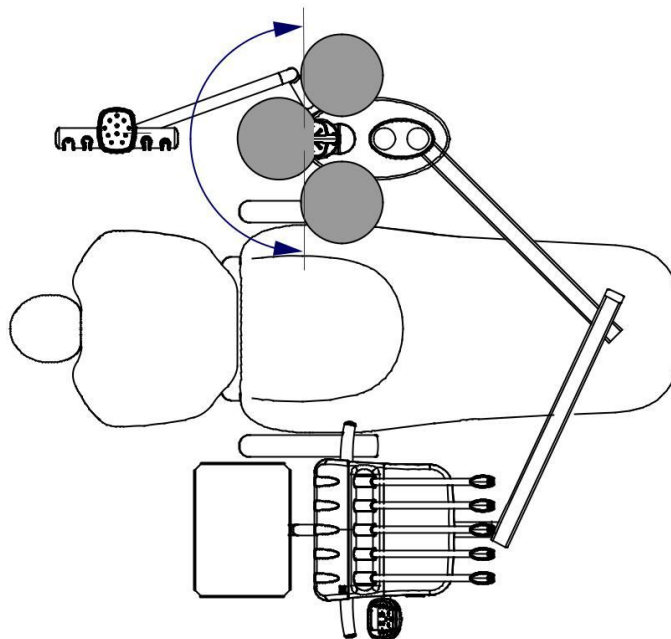
Fig.:movimiento giratorio complete de la escupidera: 180°
Fig.: maximum swivel range of spittoon bowl: 180°

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Spittoon bowl rinsing

The spittoon bowl rinsing starts when pressing key **"spittoon bowl rinsing"** and stops automatically after a programmed time.

Premature stop during bowl rinsing is possible by pressing the key once more.

Programming of spittoon bowl rinsing time:
(This is only possible at the dentist keyboard.)

- press key  longer than 5 seconds
→ the indicator LED lights
- press key  shortly
- when the desired rinse time is reached, press key  again to finish the spittoon bowl rinsing
→ indicator light goes off and the programming mode is terminated

Cup filler

The cup filling starts when pressing the key **"Cup filler"** and stops automatically after a programmed time.

Premature stop of cup filling can be released by pressing key **"Cup filler"** once more.

The recommended filling level is two thirds of the glass or plastic cup. Overflowing water will run off through the cup support plate to the drain.

Programming of cup filling time
(This is only possible at the dentist keyboard.)

- press key  longer than 5 seconds
→ the indicator LED lights
- press key  shortly
- when the desired filling level is reached, press key  again
→ indicator light goes off and the programming mode is terminated

Cup fill heating

- press key  shortly
→ the indicator LED blinks
When the water reaches the preset temperature the indicator lights continuously.

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Big suction (1)

- take out the suction handpiece
→ suction starts automatically
- take back the suction handpiece to the instrument holder
→ suction will stop automatically after 3 seconds

Small suction (2) (small suction)

- take out the suction handpiece
→ suction starts automatically
- take back the suction handpiece to the instrument holder
→ suction will stop

Suction hose and suction handpieces are expendable materials.

Aqua bottle system**Important safety regulations**

The bottle is pressurized! (ca. 2.0-3.8 bar, depends on the type of device)

Don't manipulate the bottle with spiky objects!

Use only original bottles from Ritter® Concept GmbH.

Before opening the bottle is necessary to check that the switch (valve lifter) is set to **Off** (see page 36, Fig. 2).

After filling the bottle with water always screw the bottle in the correct way into the bottle holder!

The bottle must be screwed only hand-tight to the bottle support.

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	After disinfection clean the thread of the bottle and bottle holder regularly.
---	--

	If the bottle have a mechanical damage, the bottle must be replaced by a new bottle.
---	--

	For safety reasons, the use of the bottle is limited to a period of 6 months . After that the old pressure bottle must be replaced by a new one.
---	---

In non-compliance with these safety regulations the company Firma Ritter® Concept GmbH shall not be liable for any resulting damages!

Water-bottle-system

The water-bottle-system consists of:

- water bottle (for distilled water)
- switch (at the bottom of the water unit)

Change water bottle

Set the switch for bottle compressed air (1)
to OFF position („Decompress“)
(Switch is located at the bottom of water unit.)

Remove the empty bottle out of the bottle holder
(screw bottle anti-clockwise)

Fill the bottle with distilled water

Screw the filled bottle to the support

(clockwise, Do not screw too firmly!)

Set the switch for bottle compressed air (1) to ON position („Plus press“, Bottle is now under pressure)

Bottle can only be unscrewed when it is despresurized.

1	Air break switch - bottle compressed air • „Decompress“ Bottle in pressure-free mode • „Plus press“ Bottle is now under pressure
2	Toggle - water source • „Tap water“ water from water pipeline • „Distilled water“ water from bottle

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	If there are impurities in tap water, it is necessary to use the distilled water from bottle.
---	---

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Patient chair

Features and options – patient chair

- additional counter-movement of seat while moving the backrest (Trendelenburg-movement) increases the comfort and avoids the "undress effect"
- quickset programming for five chair positions
- two articulated headrest, mechanically
- left and right armrest for patient
- seamless upholstery
- Operation of patient chair can be done manually by hand
- via dentist or assistant keyboard or by foot via multifunctional foot controller
- safety switch
- Tapestry (vinil) resistant to chemical washing agents smooth surfaces that do not allow the accumulation of pollutant waste

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	When moving the back rest, the arms of the patient must not be placed behind his back! Otherwise there is a risk of squashing!
--	---

Safety switch

 When the patient chair or backrest move down, the backrest will automatically stop as for obstruct encountering and moves up for a short distance.

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Control elements

The patient chair can be controlled from the dentist keyboard or with the foot controller.

Important:

If any instrument is taken from its holder (except syringe), the control with of the patient chair is not longer possible.

Manual control via dentist or assistant keyboard:

The patient chair is controlled manually by keys

- "Backrest UP"

- „Backrest DOWN"

- "Patient chair UP"

- „Patient chair DOWN"

and can be brought into any desired position.

The arrows on the keys indicate the direction of the chair movement.

Manual control via foot controller:

The buttons of the foot controller have following functions:

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Wichtig:

Any automatic patient chair movement will be stopped by pressing any key.

Programming the chair positions:

The programming of the automatic chair positions can be done by using the dentist keyboard. Please proceed as follows:

Treatment position A

- move the patient chair manually into the desired position
- actuate and keep actuated key „Treatment position A“  on the dentist keyboard longer than 5 seconds till you can hear a single beep

(This beep indicates, that the programming of the position was successful.)

Treatment position B

- move the patient chair manually into the desired position
- actuate and keep actuated key „Treatment position B“  on the dentist keyboard longer than 5 seconds till you can hear a single beep

(This beep indicates, that the programming of the position was successful.)

Exit position

- move the patient chair manually into the desired position
 - actuate and keep actuated key „Exit position“  on the dentist or assistant keyboard longer than 5 seconds till you can hear a single beep
- (This beep indicates, that the programming of the position was successful.)

Emergency position

- move the patient chair manually into the desired position
 - actuate and keep actuated key „Emergency position“  on the dentist or assistant keyboard longer than 5 seconds till you can hear a single beep
- (This beep indicates, that the programming of the position was successful.)

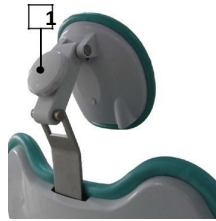
Spitting position

- move the patient chair manually into the desired position
 - actuate and keep actuated key „Spitting position“  on the dentist or assistant keyboard longer than 5 seconds till you can hear a single beep
- (This beep indicates, that the programming of the position was successful.)

Setting of linkage between spitting position and spittoon bowl rinsing:

- actuate and keep actuated key „SET“  on the dentist or assistant keyboard longer than 5 seconds till the indicator light beside the the key  is on
- actuate key „spittoon bowl rinsing“  → a signal sounds and the indicator light goes out
- spittoon bowl rinsing at spitting position is activated

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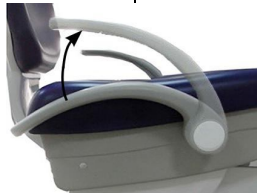
Adjust the headrest:

The double joint for wide articulation can be adjusted by the knob (1):

- rotate the clamping knob (1) anti-clockwise to loose joints
- move the headrest manually into the desired position
- rotate the clamping knob (1) clockwise to tighten it

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Move right armrest

- moving the armrest upwards, until it clicks

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Move armrest DOWN

- move the armrest upward to the highest position
- move armrest down to the lowest position

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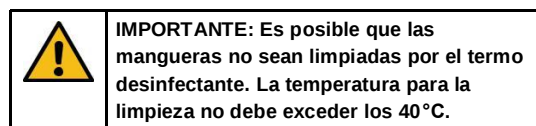
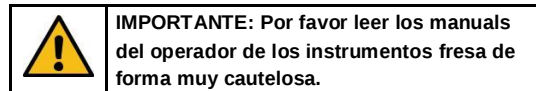
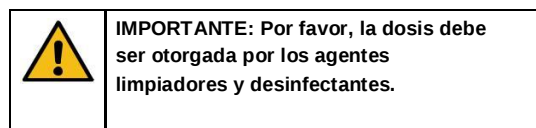
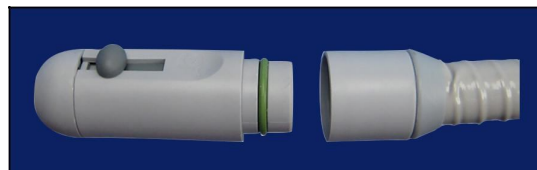
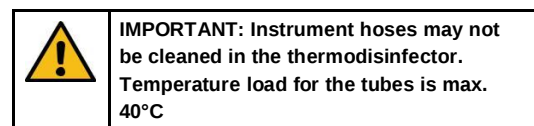
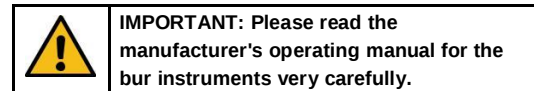
Conformity with the hygienic standards as well as good performance of the equipment are warranted by observance with the instructions to the recommendations for disinfection, cleaning and maintenance.

Bur instruments:



- Every day before the first treatment, let instruments run for approx. 120 seconds to purge all spray tubes.
- Clean and maintain turbine, handpieces and contra-angle attachments.

Suction handpieces:

Detach handpiece from hose. Clean accessible hose ends and handpieces. Lubricate O-rings with vaseline and reassemble handpiece to the hose.

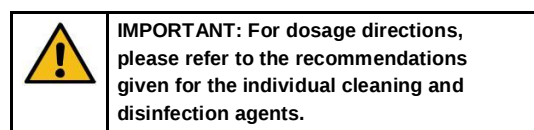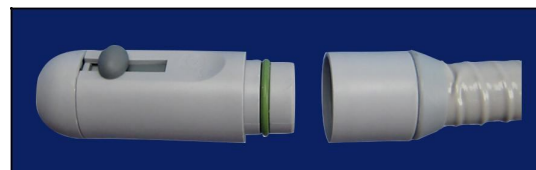Unsuited and not permitted are cleaning agents based on quaternary ammonium compositions, cleaning agents containing phenol and aldehyde as well as foam generating agents.

Small suction

Small suction should be equipped with disposable suction nozzle.

- Throw away the suction nozzle after treating each patient and replace with a new one before the treatment of next patient.

Big suction

- sterilize Big suction nozzle by high-temperature steam sterilization (134°C, 15 min)

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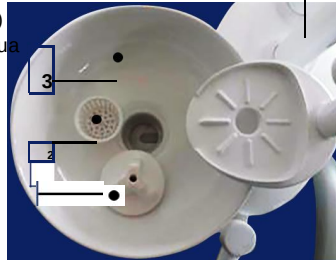
**Cleaning the filter (suction)****Saliva suction****Cleaning suction hose and suction filter**

- take out the suction handle from instrument holder
- pull out the suction hose from the suction filter
- take out the filter screen from the filter it and clean with clean water
- install suction handpiece
- install filter screen into the filter

**Cleaning pipe system:**

regularly pump clear water or special sterilized water about 30 s with recommended 1 or 2 times every day.

Water filter

The unit has water filter to ensure its normal use.

After serving a certain time, the dirt will block the filter core and affect the water flow, therefore, it is necessary to clean and replace the filter core.

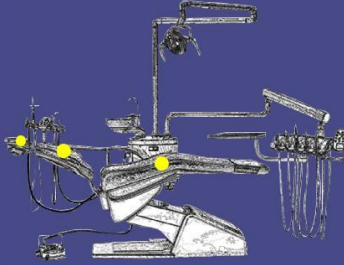
Spittoon bowl

- take out the mushroom (1) and filter screen (2) from the spittoon bowl
- sterilize the mushroom (1), filter screen (2).
clean spittoon bowl (3) and spittoon bowl holder with water and disinfect it
- install the spittoon bowl (3)
- install filter screen (2) and mushroom (1) into the spittoon bowl


Turbine exhaust air oil container

- Screw off the tank anti-clockwise, Empty the turbine exhaust air oil container (1) and replace cotton wool.
- Screw on the tank clockwise
The container is located at the bottom side of the dentist unit

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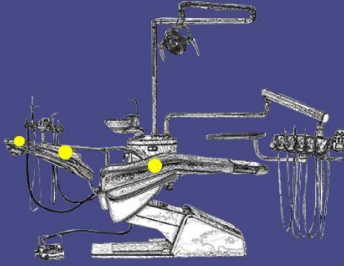Cleaning and care instructions

What	How	Wherewith	Notes
	● Clean/disinfect upholstery Wipe upholstery, wiping disinfection • Headrest upholstery • Backrest upholstery • Seat upholstery	<u>Cleaning:</u> • Soft sponge, lukewarm water, soap. • Clean heavily soiled areas with a soft brush. After being cleaned, the upholstery should be wiped dry with a cloth. <u>Disinfection:</u> We recommend applying disinfectant alcohol-free spray foam for cleaning and disinfecting imitation leather upholstery to your upholstery.	Remove soap residue with a damp sponge and dry it with softy, lint-free cloth. A cloth is not suitable for cleaning! Never use cleaning agents or household cleaners. <u>Discolouration</u> In most cases, discolouration stains that are detected immediately can be removed with the right cleaning tools and cleaner for imitation leather. Discolouration stains that are not detected immediately cannot be removed. Protect the upholsteries from direct sunlight.
	● Clean/disinfect upholstery Wipe upholstery, wiping disinfection • Backrest upholstery • Seat upholstery		

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Keep liquids away from electrical connections and components!

- Most electrical control switches ARE NOT hermetically sealed.
- When a switch get wet, it can self-activate, or perhaps short out tad damage other system components!
- Keep switches dry.
- Avoid costly equipment damage or failure.
- Please do not spray disinfectants directly at or into electrical switches and controls.
- Simply spray these solutions onto a cloth (not saturate the cloth) and wipe down the control surfaces.

Any not recommended maintenance/change by the manufacture may cause risk of injury or shock!

- Don't let more than one patient sit in the chair at one time
- Don't stand anywhere on the dental chair
- Don't operate the dental chair around any flammable liquids.

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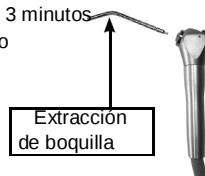
Handpiece

Handpiece is a delicate device, so it shall not be pounded or dropped.

It shall be cleaned and lubricated with special detergent and lubricant every day. As for the specific method, refer to the instruction manual of handpiece.

Oil spray lubrication shall be conducted before high temperature sterilization.

Please pay attention to the operating manual of the hand piece!

Operating lamp „Standard light LED“

Handle

- Changing
→ see description on page 27.
- Cleaning/Disinfection:
→ neutral cleaning solution, alcohol, alcohol-based disinfectants

Lamp housing

- Cleaning/Disinfection:
→ neutral cleaning solution, alcohol, alcohol-based disinfectants

Operating lamp „SunLite“

- → see instruction for use of operating light „SunLite/Polaris“

Cannula of Syringe

- sterilization in steam sterilizer: at 134 °C for 3 min
(to removing the nozzle, press spigot ring (see page 25) and remove the nozzle for sterilization)

Suction handpiece

- sterilization in steam sterilizer: at 134 °C for 3 min

Silicone Mat for instrument tray

- sterilization in steam sterilizer: at 134 °C for 3 min

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Overall failure

Failure	Troubleshooting
Failure 1: After power-on, the overall system makes no response.	1. Check whether AC 220 power supply output is in good condition. 2. Check whether the 10A fuse at power switch is broken.
Failure 2: After power-on, no water or air is supplied to the system.	Check water source change-over, pressurizing/pressure relief shift is in good position; • when water source shift is in tap water position, the pressurizing/pressure relief shift should be in relief position; • when water source shift is in distilled water position, the pressurizing/pressure relief shift should be in pressurizing position.

Common Failures

Failure	Troubleshooting
Failure 1: Operating chair fails to ascend or descend	Check whether there is AC 24V voltage input at J6 port of chair control board. 1. If there is AC 24 V voltage input at J6 port, check whether the two yellow keys can operate to achieve ascent/descent of chair; if not, the control board is likely to have damaged, replace the control board to remove the problem. 2. If there is not AC 24V voltage input at J6 port, check whether the chair power supply plug on flat cable is in good connection; if not, tighten the screw. 3. If there is AC 24V voltage input at J6 port, check whether the two yellow keys can operate to achieve ascent/descent of chair; if not, check whether spittoon protection switch is triggered by using the following method: Toggle the switch to the left to close it, and the chair can ascend or descend; toggle the switch to the right to open it, and the chair cannot ascend or descend.
Failure 2: Operating chair ascend or descend automatically	In case that operating chair ascend or descend automatically, perform troubleshooting from the following aspects: 1. Assistant Section Touch Control Board: The assistant section control board may be corroded due to damp absorption or entry of water, and thereby cause malfunction of the function keys. Inspection method: Disconnect the communication cable terminal on the control board, operate the machine by using pedal and treatment platform to check whether the failure continues; if the failure is removed, it indicates that the control board is damaged, and replace it; if the failure continues, re-connect the communication cable terminal, and go to the next step. 2. Treatment platform touch control board: The treatment platform control board may be corroded due to damp absorption or entry of water, and thereby cause malfunction of the function keys. Inspection method: Disconnect the communication cable terminal on the control board, operate the machine by using pedal and treatment platform to check whether the failure continues; if the failure is removed, it indicates that the control board is damaged, and replace it to remove the problem.
Failure 3: Water heater fails.	Inspections: 1. If water heater fails, check whether there is AC 23 V voltage output at J13; if yes, the heater may be broken and replace it; if no, the side case control board or the heater is damaged; if heater cannot be disconnected, and J14 is always closed, the heater may be broken and replace it. 2. If there is no voltage output at J13, check whether fuse F4 is broken; if yes, replace it with a 6A fuse.

Failure	Troubleshooting
Failure 4: Handheld sprayer operates normally but no water or air is sprayed out.	Inspections: 1. First check whether the sprayer operates currently in dry grinding mode where no water or air is sprayed by operating the change-over switch on it. For more details, see the operating instructions. 2. Check the water/air hose line is in good connection in instrument panel. If the hose line is in good condition, then check whether there is DC 24 V output at water/air circuit solenoid valves; if yes, the solenoid valve may be broken, and replace solenoid valve; if not, measure voltage output at reserved port J33, if there is a voltage output, replace treatment platform control board, if not, inspect the power supply in side case. 3. Foot-operated micro switch: Check whether the foot-operated micro switch resets normally; if not, replace the micro switch. 4. Replace side case control board.
Failure 5: Program chair fails to store data	Inspections: Generally, failure of program chair storage function is resulted from damage to angle sensor, replace the corresponding angle sensor.
Failure 6: There is water or oily dirt in air hose.	Solution: Check air supply source and air filter, clean or replace relevant components.

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- Check the electric supply every month.

Water pressure and air pressure must be in accordance with the unit requirement.

Keep the hoses smooth and unblocked.

- Check the chair's movement and control panel.

When required

- Water inlet filter:
cleaned or replaced the water filter element, if:
 - the filter is used after a year
 - the filter collector is contaminated
 - the water coming out becomes turbid

Replace filter element

- open the water unit
- remove the water filter
- remove the filter element anti-clockwise
- take off the sealring and the collector successively
- cleaning or replacing the filter element
- put in the collector and the sealring
- insert the filter element anti-clockwise
- check the sealring for leakage
- close the water unit

Automatic self test:

- switching-off the dental unit at main switch (control indicator of the power switch lights off)
- and press button  and keep it and switching-on the dental unit at main switch
- release the button  after an indication of beep

→ the patient chair will move up and down automatically

→ operating lamp will light

→ the heater will start

Press any key to stop the automatic self test.

Reset the programmed chair positions to factory settings:

- press key  for 5 seconds
(An indication beep sounds, the chair will first move up to the highest position then down to the lowest position. Now all programmed chair positions are reset to factory settings.)

Ultimate Comfort

Modelo: Smart


C20-0094
1801ES-EN


Apéndice


Appendix

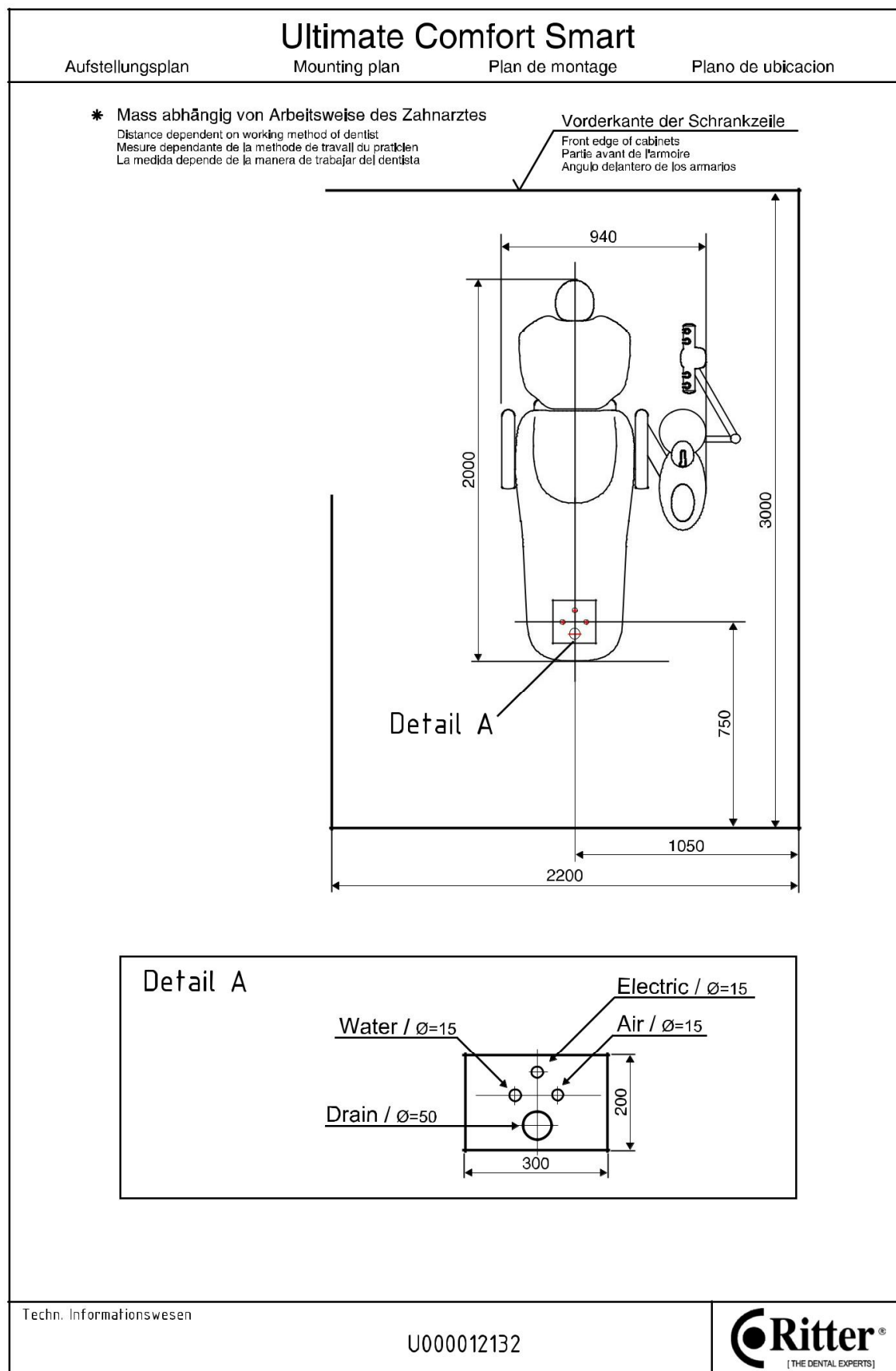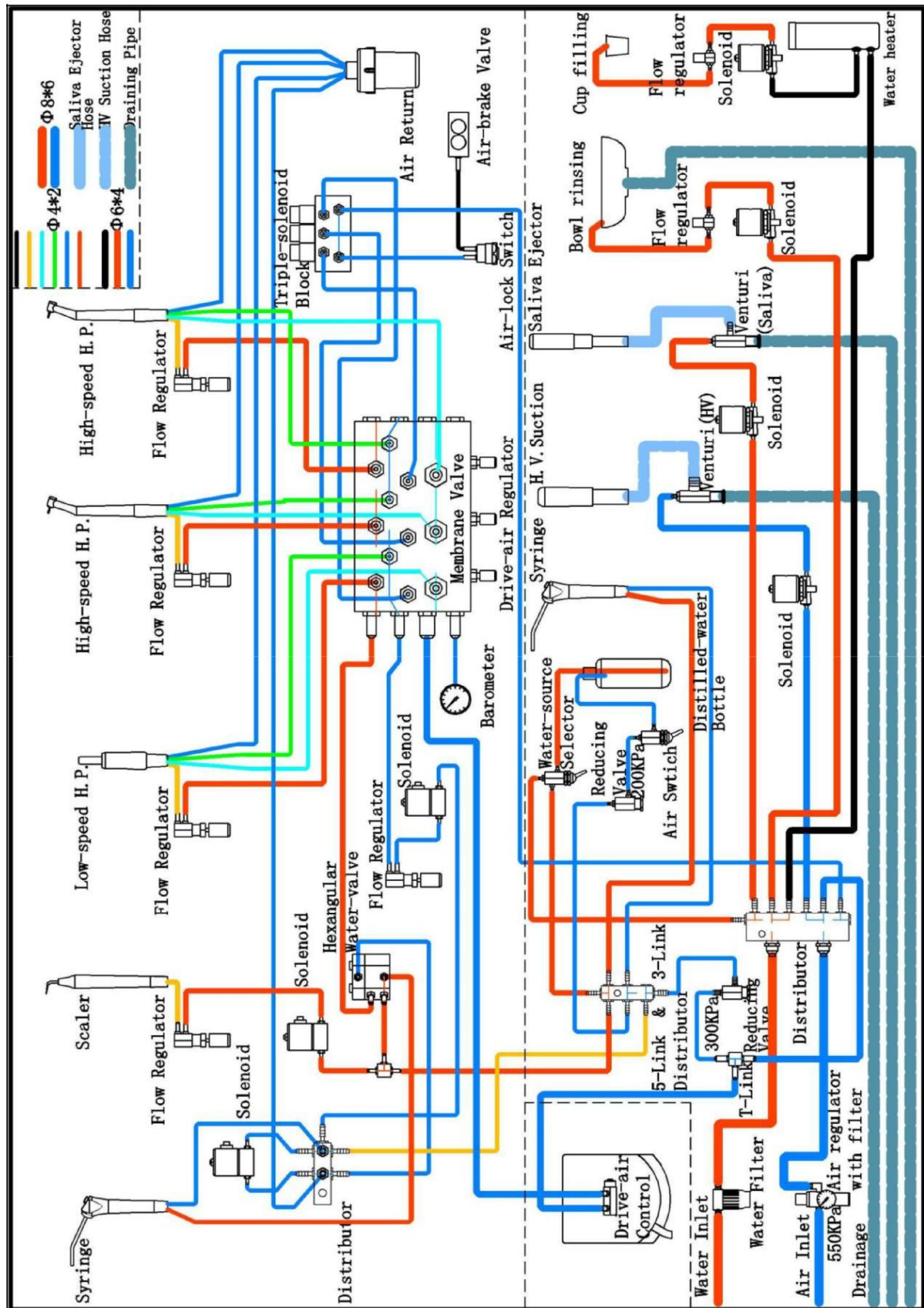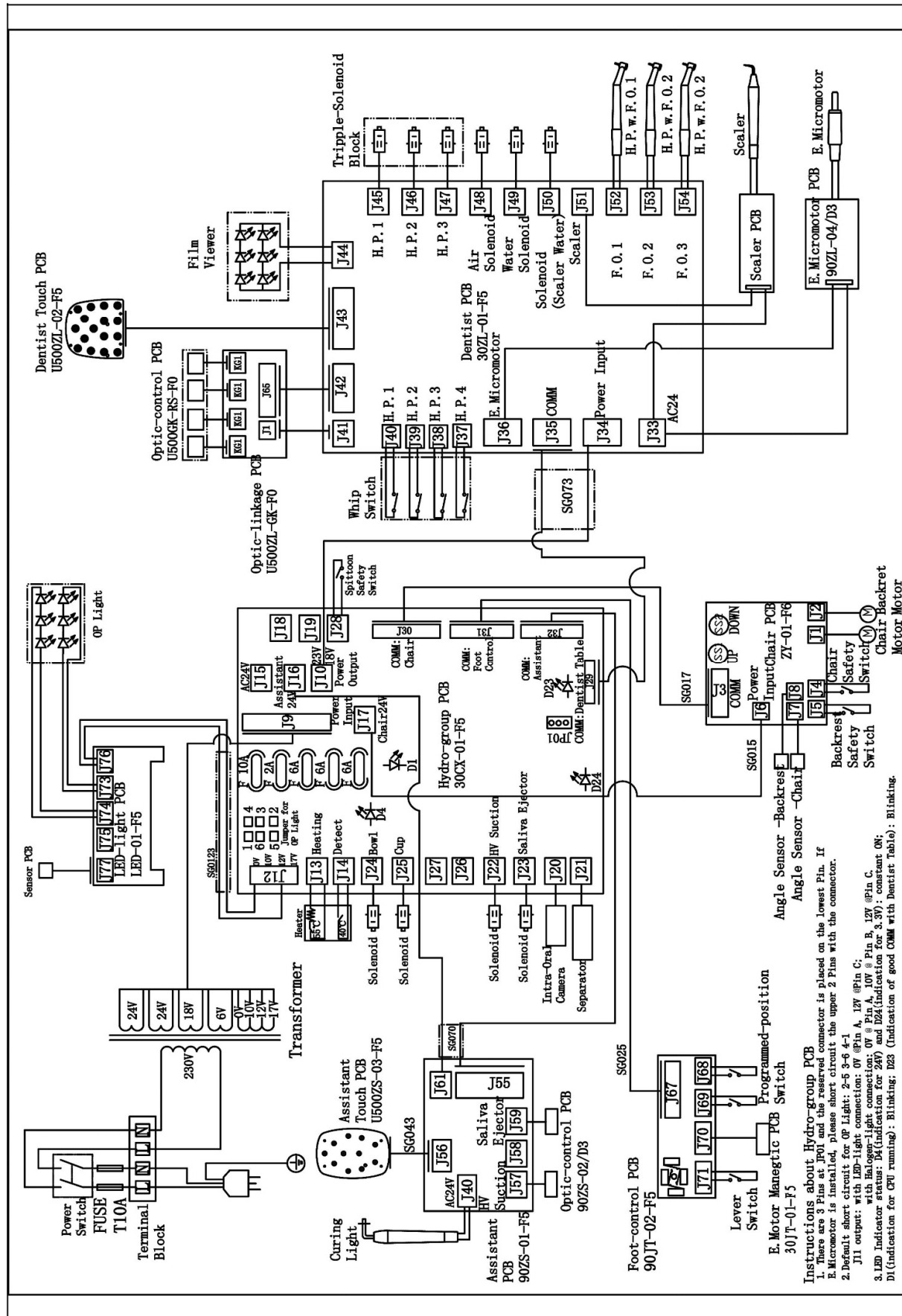CE 0044

Inspection protocol for safety inspection according DIN EN 62353			
Client (dental practice)		Contractor	
Device		Used measurement device	
Device type: Dentale Behandlungseinheit		Device type: Electrical Safety Tester	
Manufacturer: Ritter Concept GmbH		Manufacturer:	
Type:		Type:	
Serial No:			
Netzspannung:V		Insulation class: 1	
Safety inspection according: EN 62353/VDE 0751		Type of applied parts: Typ BF	
Visual inspection			
Protective earth conductor not damaged (for insulation class 1 only)			
Cover and mechanical parts not damaged			
Insulating parts not damaged			
Device connecting cables including plugs without defects			
Labels exist or completed, nameplate correctly			
Others			
Measurement			
	measured value	limit value	O.K./not O.K.
Protective earth resistance		<0,3 Ω	
Insulating resistance		>2 MΩ	
Test voltage		~500 V	
Device leakage current (Equivalent device leakage current)		<10 mA	
Encl. leakage current (Housing leakage current)		<5 mA	
Leakage current of application part (Equivalent patient leakage current)		<5 mA	
Test voltage leakage current (nominal Voltage ±10%)		min.: -10% U _{Netz} max. +10% U _{Netz}	
Functional testing			
Function and safety testing without defects			
The unit has safety deficiencies:			
Short-circuit L-N			
Body circuit L/N-SL			
Test cycle:..... months			
Next test date:.....			
Tester:			
Place, Inspection date		Signature	

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Guidance and manufacturers declaration - electromagnetic emission		
The Ultimate Comfort is intended for use in the electromagnetic environment specified below. The customer or the user of the model Ultimate Comfort should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions according CISPR 11	Group 1	The Ultimate Comfort uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. The Ultimate Comfort system is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions according CISPR 11	Class B	
Harmonic emissions according IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions according IEC 61000-3-3	Compliant	

Guidance and manufacturers declaration - electromagnetic immunity			
The Ultimate Comfort is intended for use in the electromagnetic environment specified below. The customer or the user of the Ultimate Comfort should assure that it is used in such an environment.			
Immunity tests	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) according IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst according IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV for power supply lines ±1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge according IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	±1 kV differential mode ±2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Interruptions and voltage variations on power supply input lines according to IEC 61000-4-11	<5 % U_T (>95 % dip in U_T for 0.5 cycle) 40 % U_T (60 % dip in U_T for 5 cycles) 70 % U_T (30 % dip in U_T for 25 cycles) <5 % U_T (>95 % dip in U_T for 5 s)	<5 % U_T (>95 % dip in U_T for 0.5 cycle) 40 % U_T (60 % dip in U_T for 5 cycles) 70 % U_T (30 % dip in U_T for 25 cycles) <5 % U_T (>95 % dip in U_T for 5 s)	Mains power quality should be that of a typical commercial or hospital environment. If the user of the Ultimate Comfort requires continued operation during power mains interruptions, it is recommended that the WED-3000 B Ultrasound Scanner be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field according IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
REMARK U_T is the a. c. mains voltage prior to application of the test level.			

Guidance and manufacturer's declaration – electromagnetic immunity			
The Ultimate Comfort is intended for use in the electromagnetic environment specified below. The customer or the user of the Ultimate Comfort should assure that it is used in such an environment.			
Immunity tests	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
			Portable and mobile RF communications equipment should be used no closer to any part of the Ultimate Comfort, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
			Recommended separation distance
Conducted RF according IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	$d = \left[\frac{3,5}{V_1} \right] \sqrt{P}$
Radiated RF according IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	$d = \left[\frac{3,5}{E_1} \right] \sqrt{P}$ 80 MHz to 800 MHz $d = \left[\frac{7}{E_1} \right] \sqrt{P}$ 800 MHz to 2.5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^a Interference may occur in the vicinity of equipment marked with the following symbol:
REMARK 1 At 80 MHz and 800 MHz, the higher frequency range applies. REMARK 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Ultimate Comfort is used exceeds the applicable RF compliance level above, the Ultimate Comfort should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the Ultimate Comfort.			
b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

Recommended separation distances between portable and mobile RF communications equipment and the Ultimate Comfort				
The Ultimate Comfort is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Ultimate Comfort can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Ultimate Comfort as recommended below, according to the maximum output power of the communications equipment.				
Rated maximum output power of transmitter P in W	Separation distance according to frequency of transmitter l in m			
	150 KHz to 80 MHz	80 MHz to 800 MHz	800 MHz to 2.5 GHz	
	$d = \left[\frac{3,5}{V_1} \right] \sqrt{P}$	$d = \left[\frac{3,5}{V_1} \right] \sqrt{P}$	$d = \left[\frac{7}{E_1} \right] \sqrt{P}$	
0,01	0.12	0.12	0.23	
0,1	0.38	0.38	0.73	
1	1.20	1.20	2.30	
10	3.80	3.80	7.30	
100	12.00	12.00	23.00	
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.				
REMARK 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. REMARK 2: These guidelines may not be applicable in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.				

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Terms of Guarantee

You are now owner of a high quality Ritter product. Thank you very much for your trust in Ritter Concept GmbH and would like to maintain this! You have the warranty right of 12 months. We point to the warranty terms in our General Terms and Conditions.

We warrant, that our products are free of faults by the time of handover to the supplier. In particular, we warrant, that the production is taking place based on the acknowledged technical rules and the relevant regulations. We also warrant, that the product characteristics are available, which are determined in the contractual documents. In line with the warranty we perform a remedy of defects, when substances, parts or performances are identified as flawed or contrary to contract. The warranty starts with the date of delivery. Deficiencies are to be communicated with Ritter Concept GmbH immediately after receipt of goods and immediately after discovery. The customer is permitted to send back products without prior approval of Ritter Concept GmbH. The customer has to select the most economic way of transport.

No warranty claims exist in the following cases:

- using the product outside of or contrary to its intended use, or operating the product outside of its specifications
- failure to observe the instructions on use, maintenance or installation
- improper handling or neglected maintenance --
- use of non-approved spare parts or accessories
- product modifications or conversions performed by customer or third parties
- usual wear and tear or other changes to or deterioration of the product attributable to:
 - external influences (e. g. knocks, blows, mechanical impacts)
 - water, fire
 - improper method of storage, handling or installation
 - unusual ambient conditions, especially conditions upon receiving or operating conditions at the installation location or force majeure
 - calcification or corrosion resulting from neglected maintenance or improper handling
 - contaminants in the air and water supply
 - unusual or impermissible chemical or electrical influences
 - soiling, discolouration or wear and tear on padding

The warranty as a rule excludes parts subject to normal wear, especially lamps, glass and rubber components and cables as well as plastic parts and surfaces regarding their colour-fastness.

All of our obligations under the warranty will be extinguished if the delivered goods are modified, improperly handled or processed. We do not warrant co-supplied or integrated third-party products. However, if requested we will assign to the customer our warranty claims against the vendor of the third-party product.

In order to activate the warranty, it is necessary to send a copy of the record of delivery and the warranty card to us following installation.

CE 0044