

Fi-P

Fi-G:Time for 0°C-230°C (S)

Woodpecker	Korean B brand	Chinese C brand	Chinese D brand
15	24	51	56


0.2 second heating time
High-performance conversion system ensures the efficient cutting



0.35mm pre-bend tip
Thinner tips can penetrate deep into the narrow root canal, providing you a better treatment effect.



High precision temperature control
Constant temperature prevents scalding

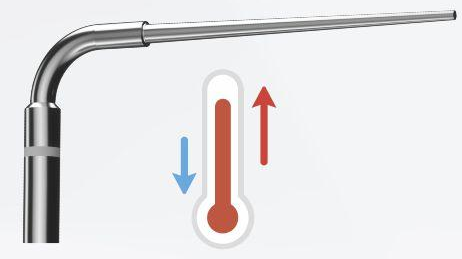

4 working temperatures
150°C, 180°C, 200°C, 230°C can match gutta-percha of different brands on the market.



4 hours of endurance
Large capacity battery & special heating system enable 1500 operations with a full charge.



Type	diameter	Taper
WP3504	0.35mm	0.04
WP4004	0.40mm	0.04
WP4504	0.45mm	0.04
WP5004	0.50mm	0.04
WP5506	0.55mm	0.06
WP5508	0.55mm	0.08
WP5510	0.55mm	0.10
WP6012	0.60mm	0.12
WP5508L	0.55mm	0.08

Fi-G

Quick heating within **15 seconds** reduces time for waiting and increase your working efficiency.




Prevent gutta-percha leakage
360 rotatable needle
Optimized structure, 360° uniform rotation, effectively avoiding gutta-percha leakage.



0.5mm injection needle
Easy to reach the apical 3mm of canal brings better filling.



Pushing ram with anti-return structure
Mechanical-locking structure, effectively preventing the pushing ram from returning and avoiding the generation of cavities and bubbles.

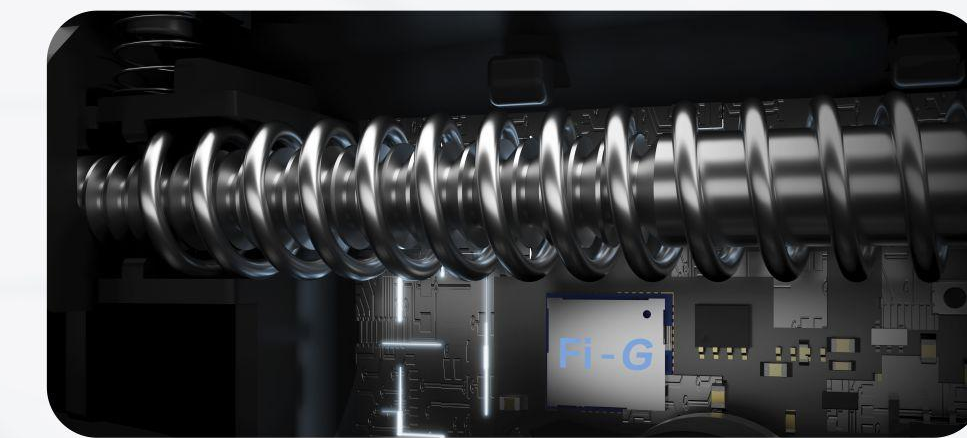

Type	Size	Length
20G 22mm	20G	22mm
20G 24mm	20G	24mm
20G 28mm	20G	28mm
23G 24mm	23G	24mm
23G 28mm	23G	28mm
25G 24mm	25G	24mm

01 According to the prepared root canal, choose a proper chief gutta-percha tip.

02 Choose proper tip for Heating and Packing Instrument, place the tip into the root canal and keep it 3-5mm away from the apical foramen, and use a pad to mark the length. At the same time, choose a plugger of the same size.

03 Dip the sealing material with the chief gutta-percha tip, and then put that gutta-percha tip into the root canal.

04 Put the tip of Heating and Packing Instrument into the root canal, press heating button for heating, cut off the redundant part of gutta-percha and take it out.

05 Put the tip into the root canal, press heating button for heating, and push the tip to soften the gutta-percha until the tip reach a place where is 3-5mm away from the apical foramen.

Root Canal Filling Procedures

- Heating and Packing Instrument
- Hot Melting and Filling Gun
- Pluggers

06 Release the heating button to stop heating, but keep pressing the gutta-percha in the root canal with tip for about 10s.

07 Press heating button for about 1s, and rapidly take out the gutta-percha 3-5mm away from the apical foramen.

08 Use pluggers to impact the injected gutta-percha in the root canal.

09 Get the needle of Hot Melting and Filling Gun touch the surface of gutta-percha in the root canal, and stay there for 3s to soften the gutta-percha.

10 Inject the melted gutta-percha into the root canal, and withdraw the needle at the same time. (Do not fully fill the root canal at only one time. It is recommended to fill it in 3 times, or a single injection should not exceed 4mm.)

11 Use pluggers to impact the injected gutta-percha in the root canal.

12 Repeat Step 9 to Step 11 until the root canal filling is finished.



Guilin Woodpecker Medical Instrument Co., Ltd.
Information Industrial Park, Guilin National High-Tech Zone, Guilin, Guangxi, 541004 P. R. China
Sales department: 0773-5873196/2350599
Fax: 0773-5822450
E-mail: woodpecker4@glwoodpecker.com
Website: http://www.gjzmm.com, http://www.gjwoodpecker.com
ZMM-KC-006

DISTRIBUTED BY:



Fi-G Fi-P

Fi-G enables the shortest heating and melting time within 15s



Gutta-percha Obturation System

Fi-G

0.5mm injection needle
Prevent gutta-percha leakage
360 degree rotatable injection needle

15 seconds fast heating

Pushing ram with anti-return structure

Fi-P

0.2 second fast heating

0.35mm pre-bend tip